



### DON'T FORGET...

- Aseptic technique and proper skin preparation of the site are essential.
- Discard BD Eclipse™ Needle in the nearest sharps collector. Follow your institution's policy for safe disposal of all medical waste.

## BD Eclipse™ Product Offerings

BD Eclipse™ Needles		
CATALOG #	GAUGE/NEEDLE	PACKAGING
305761	25g 1"	100/Box – 1,200/Case
305762	23g 1"	100/Box – 1,200/Case
305763	22g 1-1/2"	100/Box – 1,200/Case
305764	21g 1"	100/Box – 1,200/Case
305765	21g 1-1/2"	100/Box – 1,200/Case
305766	18g 1-1/2"	100/Box – 1,200/Case
BD Eclipse™ Needles with Syringes		
CATALOG #	GAUGE/NEEDLE	PACKAGING
1ml BD Luer-Lok™ Syringe with Detachable Needle		
305780	25g 5/8"	50/Box – 300/Case
3ml BD Luer-Lok™ Syringe with Detachable Needle		
305781	25g 5/8"	50/Box – 300/Case
305782	23g 1"	50/Box – 300/Case
305783	22g 1-1/2"	50/Box – 300/Case
5ml BD Luer-Lok™ Syringe with Detachable Needle		
305785 <sup>1</sup>	22g 1-1/2"	50/Box – 300/Case
10ml BD Luer-Lok™ Syringe with Detachable Needle		
305786 <sup>1</sup>	22g 1-1/2"	50/Box – 300/Case
BD Eclipse™ Blood Collection Needles		
CATALOG #	GAUGE/NEEDLE	PACKAGING
BD Vacutainer™ Eclipse™ Blood Collection Needle		
368607	21g 1-1/4"	48/Box – 480/Case
368608	22g 1-1/4"	48/Box – 480/Case
BD Vacutainer™ Pronto™ Quick Release Needle Holder		
368871		20/Box – 100/Case
BD Vacutainer™ Nestable Needle Disposable Container		
366236		100/Case
BD SafetyGlide™ Needle & Syringe		
CATALOG #	GAUGE/NEEDLE	PACKAGING
Insulin Syringe with Permanently Attached Needle		
305930	1ml 28g 1/2"	100/Box – 400/Case
305932	1/2ml 28g 1/2"	100/Box – 400/Case
305935	3/10ml 29g 1/2"	100/Box – 400/Case
1ml Tuberculin Syringe with Permanently Attached Needle		
305945	27g 1/2"	100/Box – 400/Case
305946	26g 3/8" (IB)	100/Box – 400/Case
1ml Allergist Tray with Permanently Attached Needle		
305950	27g 1/2"	25/Tray – 1,000/Case
305951	26g 3/8" (IB)	25/Tray – 1,000/Case
1ml Syringe with Detachable SafetyGlide Needle		
305903	25g 5/8"	50/Box – 400/Case
Needle		
305901	25g 5/8"	50/Box – 500/Case

<sup>1</sup> Available Summer 2002



BD Medical Systems  
1 Becton Drive  
Franklin Lakes, NJ 07417  
www.bd.com

BD, BD Logo and all other trademarks are property of Becton, Dickinson and Company. ©2002 BD. BD 5/02-0425



Indispensable to human health

# Safety made simple.

## BD Eclipse™ Needle

# Step-by-Step Safety

- ### 1 Aspiration

  - Aspirate medication into syringe as per your usual technique.
  - Reposition safety cover, if necessary, so it does not obscure injection site.
- ### 2 Injection

  - Administer injection according to protocol.
  - For user convenience, the needle "bevel-up" position is oriented to safety cover. (Bevel-up = Safety cover up)
- ### 3 Activation

  - Immediately following injection, activate safety cover by centering thumb or forefinger (of hand holding syringe) on finger pad area of safety cover, then pushing cover forward until you hear/feel it lock.

**For use with Luer Lock syringe only.**

## EASE OF USE

- Single-handed activation
- Minimal change in technique
- No hard surface required for activation

## SAFETY

- Safety activation immediately follows injection
- Audible click indicates safety activation
- Needle remains locked inside activated cover

## PATIENT CARE

- Activation occurs after needle is withdrawn; no negative impact to the patient
- BD PrecisionGlide™ Needles minimize injection pain and patient discomfort

## EFFICIENCY

- Includes 18 gauge needle for filling and other "large bore" applications
- Needle-based design facilitates standardization and SKU reduction

# BD Eclipse™ Needle.

## Safety within your reach.

### PRIOR TO ACTIVATION



### AFTER ACTIVATION

Needle remains locked inside activated cover



**Use BD SafetyGlide™ Syringes  
for all your insulin, TB and allergy needs.**

## Committed to meeting your specific safety needs

The BD Eclipse™ Needle marks an addition to the growing family of BD safety injection products. The full range of BD safety-engineered devices includes syringe-based and needle-based designs to meet your specific needs. In addition to BD Eclipse™, BD offers BD Safety-Lok™, the world's first safety syringe, the BD SafetyGlide™ Needle, and the BD SafetyGlide™ Syringe for small dose applications.

With the introduction of the BD Eclipse™ Needle, BD now offers one safety technology for both your injection and blood collection needs.



This full array of safety-engineered injection devices demonstrates our commitment to providing you with a safety device best suited to fit your needs. At the same time, BD continues to invest in the development of safety technology that will provide innovative products and solutions for the future.

For more information call  
**1.888.237.2762**